

QuikRead® CRP

Brief description of the test procedure




1

Preparation:
Read the card.



2

Dispense 1 ml of buffer into
the cuvette.

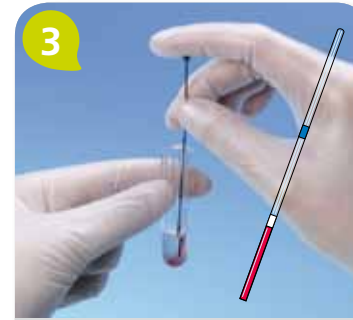

3

Add 20 µl of whole blood.



4

Put the cap on and mix gently
(not upside-down). Wait until the
whole-blood solution clears up as
a result of haemolysis. Put the cu-
vette into the measurement well.



5

Blank measurement max. 40
seconds. Add the CRP-reagent
by pressing down the inner
part of the cap.



6

Take out the cuvette and mix
vigorously (back and forth) to
dissolve the reagent.



7

Put the cuvette back into the
measurement well. The CRP
result is displayed within 1
minute.

For
more detailed
information,
please consult the
instructions
for use.