Clean Card PRO

Intended use and indication for use

Aidian Clean Card® Pro test is for monitoring the effect of cleaning processes on surfaces where protein contamination is possible.

Background

Protein residues on surfaces indicate insufficient cleaning. Insufficient cleaning may pose a threat to health and even cause economic losses in the form of product loss because the unclean surface supports growth of pathogens.

Test principle

Each test device contains a reagent pad that is impregnated with reagents. The reagent pad is placed under the perforated part of the device.

When proteins are present on the surface and brought into contact with the reagents by wiping a dampened surface with the test pad, a visually noticeable colour change occurs. The colour changes from pale yellow to green or blue. When the surface is clean, the colour remains pale yellow.

Contents of the kit

Clean Card PRO	Cat. No. 133148	Cat. No. 133149
Test device	25	50
50 ml plastic bottle	1	–
Spray cap for the plastic bottle	1	–
Model chart	1	1
Instructions for use	1	1

The plastic bottle should be filled with clean water that is cold or at room temperature. Drinking water is suitable for this. All other materials required for testing are supplied in the package.

Warnings and precautions

Health and safety information

- The device does not contain components that may pose a threat to health.
- A clean result does not exclude the possible presence of infective agents because the test does not detect microbes directly.
- A material safety data sheet is available from the manufacturer upon request.
- Disposal: see "Recommendations for disposal".

Analytical precautions

- Do not use the reagents after the expiry date.
- Do not touch the reaction area since it may cause an erroneous positive reaction.
- Discard those devices whose reaction area has been touched.
- To avoid contamination of the reagent pad, the plastic parts outside the reaction area of the test device should be intact.
- The devices are disposable, do not re-use.

Reagent storage and stability

Store the tests at 18...25°C in their own plastic bag. The shelf life marked on the kit's label is only guaranteed under the given circumstances.

Test procedure

- **Surface**
Test an area of 10 cm x 10 cm.
- **Sharp objects, corners and joints**
Fold the device in such a manner that the reagent pad comes into contact with the area to be tested.

Procedure

Start by filling the plastic bottle provided in the kit with clean water. Seal the bottle with the spray cap. Empty, rinse and refill the bottle when needed, at least daily.

Performing of the test

- 1 Take the device from the box: take care not to touch the perforated part of the device.
- 2 Moisten the surface you are about to test by spraying water twice on the surface.
- 3 Press the perforated part of the test pad against the surface you moistened and wipe the entire moistened area by pressing the device firmly against the surface. The moisture must be absorbed into the test.
- 4 Wait for the colour to develop for at least 30 seconds. Check the colour change in the reagent pad by comparing the pad with the model chart.
- 5 Water should only be sprayed onto the surface, not directly to the reagent pad.

Colour change from pale yellow to green or blue	→	Positive reaction: Unclean surface. Clean again.
Colour remains pale yellow	→	Negative reaction: The surface does not contain proteins in measurable amounts.

A visible colour change in one of the reagent circles is enough to indicate a dirty surface. The acceptable level of cleanliness must be set by the user. A negative test result may not be a necessary requirement in all inspection cases. A weak positive may be acceptable in certain cases.

The colour remains unchanged for approximately 30 minutes. When the pad has dried the coloured result may change and the result is no longer readable. Do not re-use. A used test does not function reliably.

Expected values

The test reacts to protein levels of 50 µg or more. The darker blue the reagent pad becomes, the larger amount of protein was detected. The acceptable level of cleanliness must be set by the user. A weak positive may be acceptable in certain cases.

Interfering factors

Some cleaning and disinfection agents cause a colour change if they are present on the surface. Even in this case, the cleaning process should be repeated. If it is desired that the disinfectant remains on the surface, the testing must be done before disinfectant is sprayed.

Limitations of the method



- Assay procedures other than those specified in these instructions for use may yield uncertain results.
- Proper sample collection is necessary for reliable test results.
- The test reacts only with protein residue.
- Should other contaminants, e.g., reducing sugars be present, they are not detected.
- Samples containing protein in concentrations below the detection limit of the test show a negative result.

Recommendations for disposal

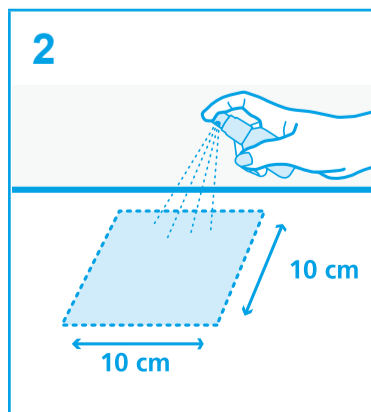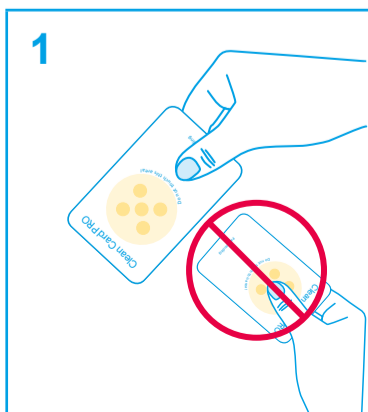The reagents supplied should not present a health hazard and used devices can be disposed of with regular waste.

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Aidian Oy