

Brief instructions



1 Remove the foil cover from the cuvette. The liquid surface should lie between the two lines marked on the cuvette. Do not touch the clear flat surfaces in the lower part of the cuvette.



2 Fill the capillary with the sample (20 μ l) up to the white stopper. Ensure that there are no air bubbles in the capillary. Wipe away any residual sample from the outside of the capillary.



3 Place the sample (20 μ l) into the buffer solution in the cuvette and dispense by pressing down the plunger. Make sure that the capillary is completely empty.



4 Close the cuvette tightly with a CRP reagent cap. Do not press down the blue inner part of the cap. Once the sample is added to the buffer, it may be held for up to two hours prior to running the assay.



5 Choose *Measure* on the display of the QuikRead go instrument.



6 Insert the cuvette in the measurement well of the instrument. The bar code should be facing you. The display shows how the measurement is proceeding.



7 When the measurement is completed, the result is shown on the display and the cuvette will rise up from the measurement well automatically. With the QuikRead go CRP+Hb test the Hb result is shown below the CRP result.

Please consult the instructions for use before performing the test.

Reagent storage



Kit box

Storage of unopened kit in 2–8°C: until the expiry date of the kit



Unopened cuvettes after opening the outer foil pouch

Storage in 2–8°C: 6 months

Storage in 18–25°C: 3 months



Reagent cap tube

Storage in 2–8°C: until the expiry date of the kit

Storage in 18–25°C: 1 month (3 months when 7,5 hours/day)



Opened cuvette

Let the buffer warm up to the room temperature before performing the test. The sample is stable in the buffer for **two hours**. Perform the test within **two hours** from opening the cuvette.



CRP Controls

(see table below)

Unopened vials in 2–8°C: until the expiry date

Opened vial in 2–8°C: up to two months

Hb Control

(see table below)

Unopened vial in 2–8°C: until the expiry date

Opened vial in 2–8°C: up to one month



Orion Diagnostica Oy
Koivu-Mankkaan tie 6 B, P.O.Box 83, FI-02101 Espoo, Finland
Tel. +358 10 4261, Fax +358 10 426 2794
www.oriondiagnostica.com, www.quikread.com

8064-01GB, 02/2016. International version. Not intended for US market.

Product	Cat. no.
QuikRead go CRP, 50 tests » with capillaries and plungers	135171
QuikRead go CRP+Hb, 50 tests » with capillaries and plungers	140068
QuikRead CRP Control 1 ml	68296
QuikRead go CRP Control High 1 ml	137071
QuikRead go Hb Control 1 ml	141154
QuikRead capillaries 20 µl, 50 pcs	67962
QuikRead Plungers, 50 pcs	67966
QuikRead go Instrument	133893

Find more information: www.quikread.com