



Uricult[®] Vet

On-site urine culture system

On-site urine culture test

The Uricult Vet dipslide system is based on two culture media for detection of microbes causing urinary tract infections in animals. One side of the plastic slide is covered with green CLED medium and the other side with reddish-brown MacConkey medium. The CLED medium is intended for determining the total microbial count. The MacConkey medium is intended for detecting gram-negative microbes. The result is interpreted by comparing the amount of bacterial growth with the Model Chart available separately. The result should be used with a complete clinical evaluation.

Urinary tract infection (UTI) accounts for significant use of antimicrobials in dogs and cats. Testing for UTI helps in making proper diagnosis to support the evaluation of the need for antimicrobials¹.

Because...

RELIABLE

- » Accurately indicates the presence of a urinary tract infection²

EASY TO USE

- » Sample, incubate and interpret

ON-SITE

- » No need to transport the urine sample
- » Gives accurate results in 16–24 hours
- » The tube maintains the integrity of the sample if further testing is needed

LONG SHELF-LIFE

- » Storage of the tubes at room temperature for up to nine months
- » Ideal for any practice size

References

1. Weese et al. Antimicrobial Use Guidelines for Treatment of Urinary Tract Disease in Dogs and Cats: Antimicrobial Guidelines Working Group of the International Society for Companion Animal Infectious Diseases. *Vet. Med. Int.* 2011. Article ID 263768.
2. For more information, please consult instructions for use.

Product list	Cat. No.
Uricult Vet 10 tests	06064
Cultura M 230 V* Incubator	129997 / 05866
Cultura M 12 V Incubator (with a car plug)	132697

*Other versions available



IT'S SO EASY TO USE!



Dip / Pour / Drop



Incubate



Interpret

For more detailed information, please consult the instructions for use.



Orion Diagnostica develops, manufactures and markets diagnostic test systems for healthcare professionals with over 40 years of experience. Building well-being is the core of our mission. Uricult® is a registered trademark of Orion Diagnostica Oy.

Orion Diagnostica Oy
Koivu-Mankkaan tie 6 B, P.O. Box 83, FI-02101 Espoo, Finland
Tel. +358 10 4261, Fax +358 10 426 2794
www.oriondiagnostica.com

8089-01EN, 02/2018. International version. Not intended for the US market.